

VIOGUARD CUBBY – UV Disinfection System

<https://www.vioguard.com/product/vioguard-cubby-plus/>

Directions:

1. Open the tray door when LCD reads *READY*
2. Carefully slide out glass tray.
3. Place your items to be disinfected on tray and slowly close the tray and door
4. Once the door is closed the UV disinfection process begins and you will notice the status bar on the LED screen change.
 - a. The status bar tracks the UVC disinfecting cycle. The bar will fill from left to right with a designation line defining the end of the Quick Clean cycle and beginning of the Deep Clean Cycle. It will remain filled until the door is opened.
 - b. Quick Clean Cycle: Average 15-20 seconds from start of process (door closure).
 - c. Deep Clean Cycle: Average 90-180 seconds from start of process.

Vioguard Cubby should only be used for personal devices such as phones, tablets, vocera badges and pagers. Absolutely NO, PPE, surgical or anesthesia equipment/instruments should be placed in the unit.

